



Medizyme for cleaning & presanitisation of all dental unit vacuum systems

Medizyme will hygienically clean and deodorise collector traps, drains, filters, hoses and pipes throughout the entire evacuation/plumbing system.

DIRECTIONS:

- 01**
Dispense 30mL of Medizyme into 5 Litres of cold water.
- 02**
Remove all solids from filters and traps.
- 03**
Place all evacuation hoses into Medizyme solution. Allow to soak for at least 2 minutes.
- 04**
Use intermittent aspiration until all solution is used.
- 05**
Clean in line screen and bowl at the vacuum unit weekly.
- 06**
Flush lines with water.

NB: Do not store mixed Medizyme. Always use entire amount of freshly mixed Medizyme solution.